


HOW TO

Connecting ri-sonic PCP-USB to a Windows PC

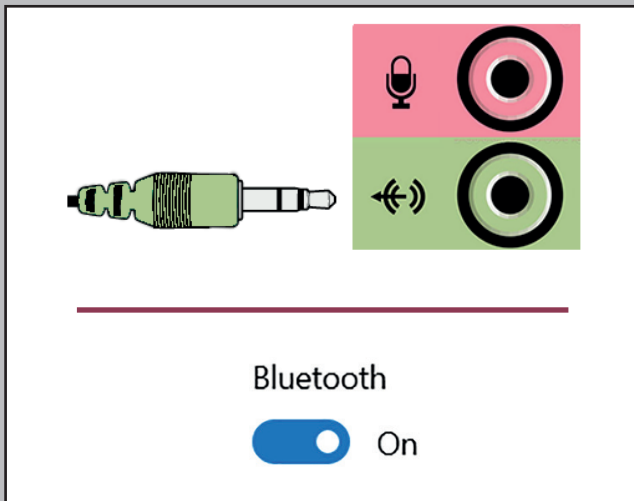


STEP 1. Turn on the PC. Connect headphones.

- **AUX headphones:** Use the green AUX input
- **Bluetooth headphones:** Go to windows settings > bluetooth> Turn bluetooth on. Add/pair headphones under Other devices.


Wait about 20 seconds, until Device is ready notification appears on the screen.




STEP 2. Connect **ri-sonic** PCP-USB on the USB port. Wait about 20 seconds, until Device is ready notification appears on the screen.

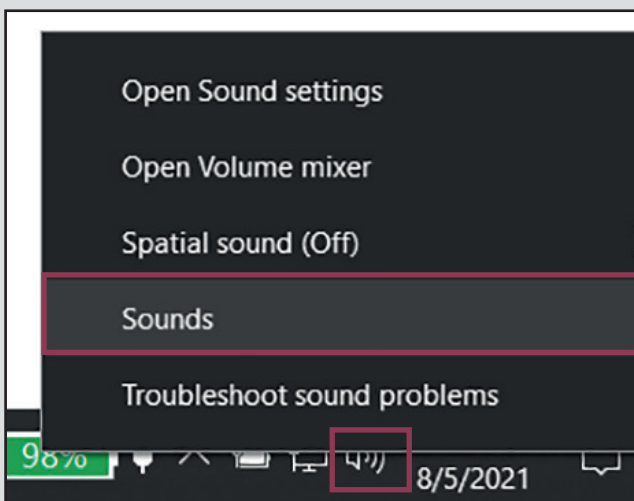


STEP 3. Find the Sound icon  in the notification area (lower right corner of your screen). Right-click the Sound icon > Click Sounds from the content menu.

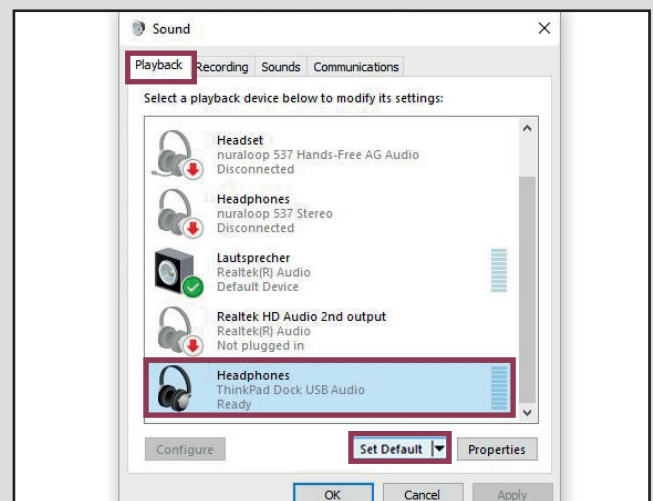


STEP 4. The Sound window will pop up. Shift to the Playback tab> Select the Headphones> Click Set Default.

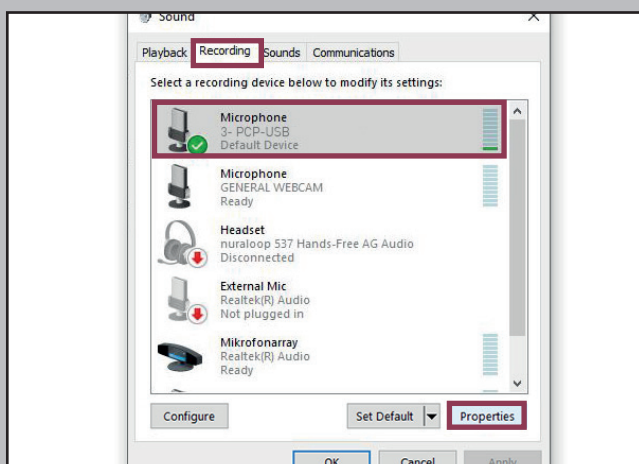


STEP 5. Shift to the Recording tab> Click the **ri-sonic** PCP-USB Microphone> Click Properties.

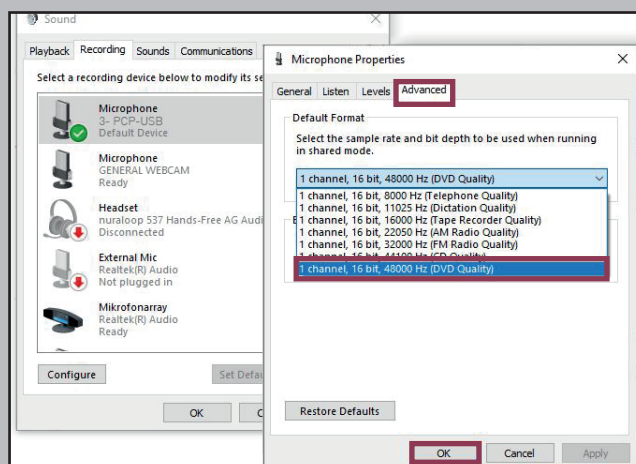


STEP 6. Microphone Properties window will pop up. Shift to the Advanced tab> Change the Microphone Default Format option to 48000 Hz (16-bit normally, 24-bit if available)> Click OK.

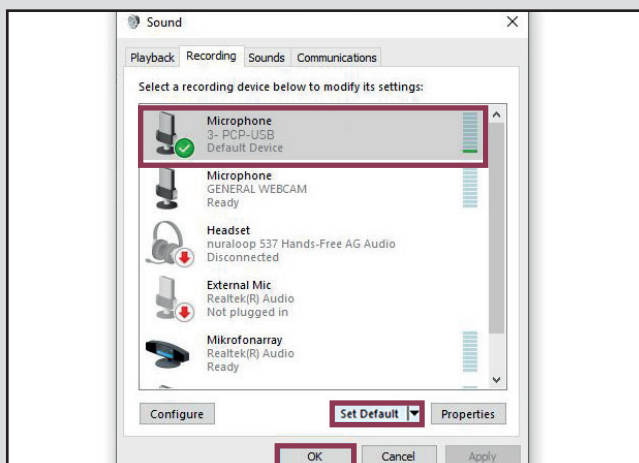


STEP 7. Microphone Properties window will disappear. Select the **ri-sonic** PCP-USB Microphone> Click Set Default> Click OK.

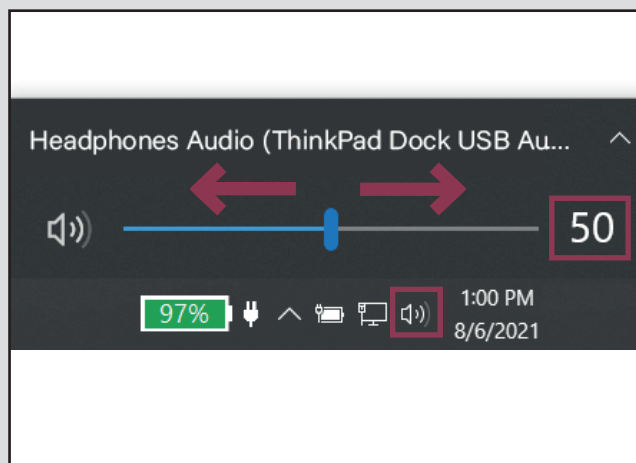

STEP 8. Click Sound icon  in the notification area. (lower right corner of your screen). Move the slider to 50% volume.

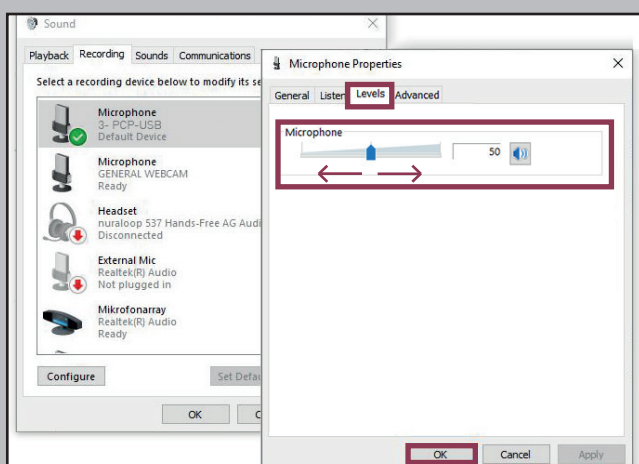

STEP 9. Open the Sound window. Shift to the Recording tab> Click the Microphone> Click Properties. In Microphone Properties window, Shift to the Levels tab> Adjust volume> Click OK. In Sound window, Click OK.

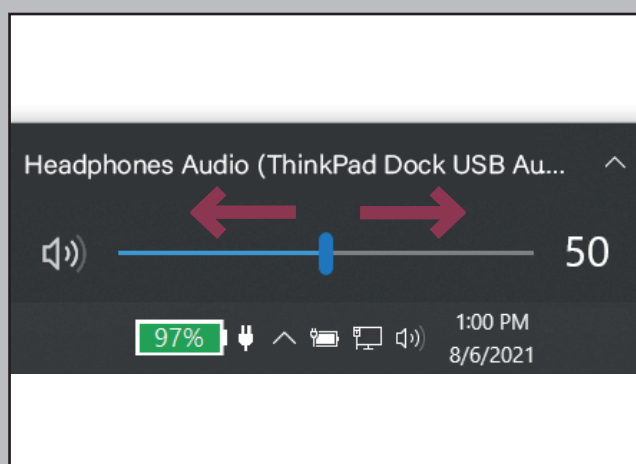

STEP 10. Overall volume is too low, adapt headphones volume once again.

Please visit **riester.de** for more information